

## MK1454-G4



MK1454-G4

- No interference with medical equipment
- Interfaces with all BENDER Clocks / Elapsed Timers ZT1490 and ZT1491
- Uses low voltage wiring (5V DC)
- Mounts to standard electrical gang box
- Connection is by screw terminal strip
- Easy to clean rugged stainless steel

### Product Description

The Remote Control MK1454 is designed as a remote for the Surgical Chronometer ZT1494.

The control buttons of the remote are mounted on a stainless steel wall plate. The MK1454 is only available in a standard 4 gang configuration.

### Operational Information

The MK1454 is able to activate the timer and to set the time of day of the Surgical Chronometer. In the timer mode a count may be interrupted, resumed and reset whenever desired. The "HOLD/RESUME" buttons of the remote are supplied with red LED's, which will flash while a count is stopped.

To set time of day, the "HOURS" and "MINUTES" buttons are both depressed. The display will start to blink. The hours can now be advanced by depressing the "HOURS" button. When the desired time is displayed the "MINUTES" button is depressed

### Ordering Information

Type	Article No.
MK1454-G4	927778

**Technical Data for MK1454-G4**

Operating voltage 5V DC  
 Max. current 20 mA  
 Operation class continuous operation  
 Ambient temperature when operating +32° F to +122° F  
 0° C to +50° C  
 when stored -13° F to +158° F  
 -25° C to +70° C  
 Connection screw terminal block  
 Max. conductor size 14 AWG  
 stranded  
 by screws  
 Mounting  
 Weight MK1454 1lb

**Safety Instructions**




Electrical equipment shall only be installed by qualified personnel in consideration of the applicable safety regulations.


**Wiring Diagram**

**TERMINALBLOCK**

\* Shielded cable

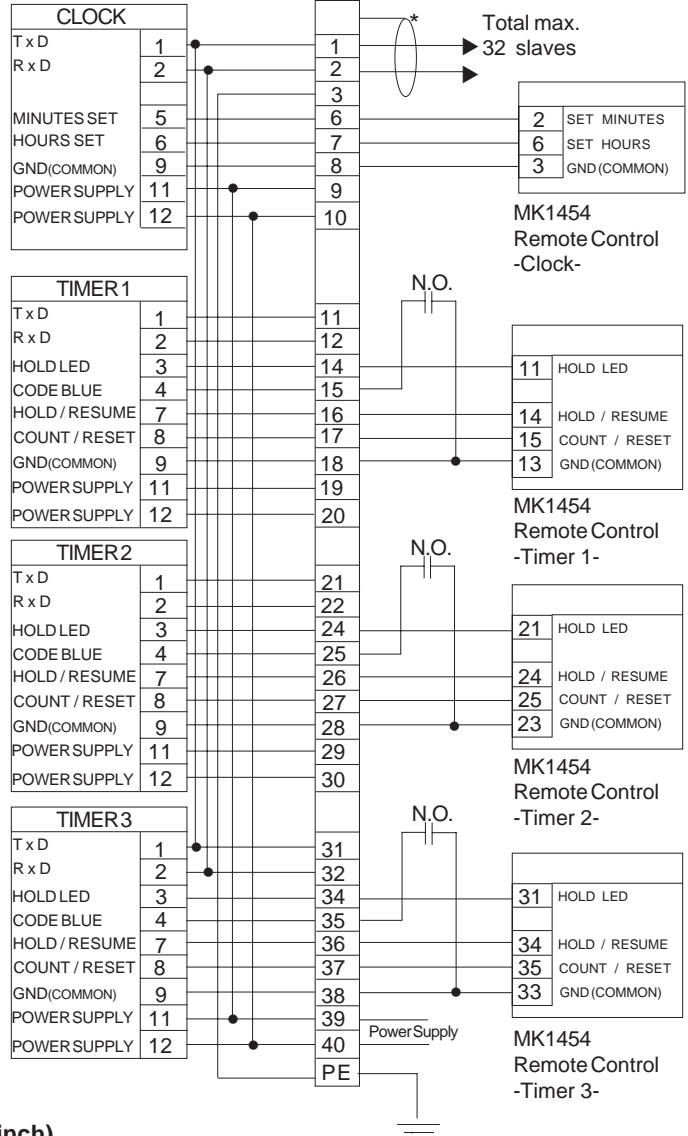

**Dimension Diagram (inch)**

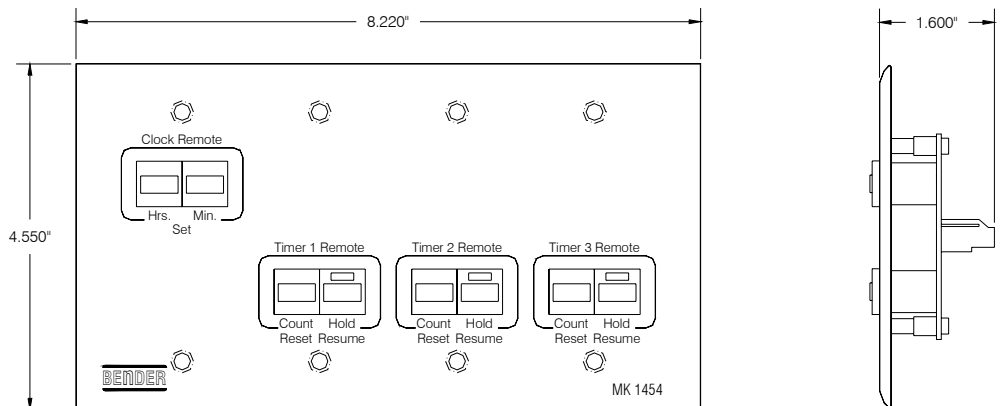

**Our Address**

**BENDER MEDICAL PRODUCTS**  
 700 Fox Chase  
 Coatesville, PA 19320  
 Phone: 800-356-4266  
 610-383-9200  
 Fax: 610-383-7100  
 E-mail: medical@bender.org

**MK1454-G4**